


BrainSee

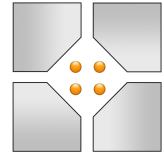

Darmiyan

## Imaging center guide for BrainSee orders

1. Acquire brain MRI sequences according to your standard protocol(s), as you would otherwise do in the absence of BrainSee.
2. Provide your routine radiologist reporting of the acquired sequences, as you would otherwise do routinely in the absence of BrainSee.
3. **In addition to your routine sequences**, acquire a “quantitative”, non-contrast, T1-weighted brain MRI with the following specifications:
  - a. 3D, T1-weighted
  - b. Non-contrast
  - c. 1 mm isotropic resolution
  - d. Either 1.5 Tesla or 3 Tesla
4. Unless specifically requested by the referring physician, no quantitative analysis is required by the imaging center on the additional, high-resolution 3D T1 sequence that was acquired. BrainSee will take care of it.
5. **Provide a copy of all brain MRI scan files (DICOM) to the patient or the referring physician, on CD, DVD, flash drive, or your physician portal.**
6. If the referring physician requests that you upload the patient’s DICOM files directly to the BrainSee platform, contact [support@darmiyan.com](mailto:support@darmiyan.com) to set up your direct access portal.